



Apix Propoli Aerosol



13.50 EUR

10 VIALS OF 2 ML

Apix® Propoli Aerosol, from Bios Line's research: ideal for cleaning and hydrating the respiratory tract and favouring the elimination of excess mucus; useful in case of exposure to pollutants, smog, smoke and dust, temperature changes and chills that may cause inflammation.

It contains a Salt with a particular composition, from the Austrian Rock salt caves, Propolis hydroglyceric extract, Grapefruit seed glyceric extract, Camomile dry extract rich in Apigenin, Eucalyptol and Thyme essential oil.

The hypertonic formulation makes it suitable for adults, children and the elderly.

WHAT IT CONTAINS

Glycerine, Sodium Chloride, Propolis hydroglyceric extract, Grapefruit seed glyceric extract, Camomile dry extract titrated with 1.5% apigenin, Thyme essential oil, Eucalyptol, Sodium phosphate dibasic dihydrate, Sodium phosphate monobasic dihydrate, EDTA, purified water as required.

HOW TO USE IT

To be used in normal or ultrasound aerosols, or directly into the nose, pressing the vial several times. Once opened, the vial must be used straight away and cannot be kept for later use.

Adults: 1 vial twice a day, pure or diluted with 1ml of saline.

Children: 1 vial once a day, pure or diluted with 1ml of saline.

Warnings: do not ingest the product; avoid exposing it to heat and sun rays; avoid contact with the eyes, in case of an accident wash with plenty of water; keep out of the reach and sight of children. In



case of delayed hypersensitivity to one or more components, avoid using the product; in case of unwanted reactions, suspend treatment and consult your doctor. Do not use the product after the expiry date printed on the packet or if this appears to have been opened or damaged. Any sediment is the consequence of the use of natural substances and does not affect the product quality. Dispose of the container responsibly. Store below 30°C.

IT IS AN EC MEDICAL DEVICE.

Read the warnings and instructions carefully.

Presidio medico: 1